

# UniCid Forte Plus

Instruments Detergent & Disinfectant

**With Three Enzymes**

## UniCid Forte Plus

**High Quality Detergent & Disinfectant with a unique formula for Medical and Dental Instruments.**

## UniCid Forte Plus

**With a very efficient concentrated enzymatic cleaner For Medical Thermal Washers & Manual use.**

**Use for surgical instruments and endoscopes.**

**contains a combination of enzymes (protease, lipase, amylase) that degrade proteins, fats, and blood residues.**

**This synergy of enzymes removes biofilm from the internal and external surfaces of the biopsy channels of endoscopes.**

**Non-Foaming.**

### CHARACTERISTICS:

**Detergent and disinfectant with three Enzymes effects.**

**Free of aldehydes and phenols.**

**Non protein fixing.**

**Non-Foaming.**

**Pleasant smell.**

**Non corrosive.**

**Biodegradable according to the classification by the OECD (Organization for Economic cooperation and development).**

### Instructions for use:

**For Medical Thermal Washer:**

**follow the manufacturing instructions.**

**In immersion bath:**

**Dilute 30 ML UniCid Forte Plus in 970 ML soft water according to the dosage recommendations. Soak all instruments and open all instruments with joints. Immerse them into the solution. Be sure they are fully covered and leave them soak for a few minutes (recommended 30 Minutes). Use it with Ultrasonic cleaner for 5 Minutes only.**

**Rinse your instruments. Dry thoroughly with a single-use paper. Your instruments are ready for sterilization.**

**Change the solution every 48 hours or when it becomes dirty.**

**Do not mix with other disinfectants.**

**The packaging must be closed tightly after use.**

**Active materials:**

**Benzalkonium chloride, Binzotriazole, Enzymes (protease, lipase, amylase), non-foaming agent, other ingredients.**

**STORAGE CONDITION:**

Store in cool and dry place

Store between +5°C and +35°C.

**Caution:**

\*Do not mix with other cleaners or disinfectants.

\*Avoid contact with eyes and skin. In case of contact rinse with plenty of water

\* Gloves usage is recommended.

**Code:3043**